

RADIOLOGY DEPARTMENT EQUIPMENT SPECIFIC TRAINING

(Angio Room)



Hospital	
Equipment Name	
Operator	
Assessor	

KNOWLEDGE	Tick
Demonstrate how to switch the unit on and perform the warm-up procedure.	
Enter patient details / use the work list.	
Be able to lower and raise the table.	
Be able to move the tube up, down, and use angles.	
Select the appropriate programme curve / program and pulse rate	
Use collimation and filters.	
Be able to magnify an image / have knowledge of dose relation to magnification.	
Use Pre-Set table positions for a "bolus chase" run	
Know what peddle screen and which peddle takes a spot image.	
Be able to review images / adjust images for contrast / brightness etc prior to archiving.	
Show how to export images and insure that images have been sent correctly.	
Be able to delete images from the hard disc / local disc	
Have knowledge of how to remove the room from action.	
In event of cardiac arrest, make table safe before commencing CPR	
Demonstrates an understanding of infection control.	
Find the emergency stop button & how to reset.	

I certify that the above person has been trained and is competent to use the equipment without supervision. I further certify that this list is not exhaustive and does not constitute comprehensive training.

Signed _____ Assessor _____ Date _____

I have received adequate training and I feel that I am competent to use this equipment without supervision.

Signed _____ Operator _____ Date _____