

RADIOLOGY DEPARTMENT EQUIPMENT SPECIFIC TRAINING

(Injector)



Hospital	
Equipment Name	
Operator	
Assessor	

KNOWLEDGE	Tick
Switch the injector on.	
Be able to load a syringe into the injector.	
Connect the injection tube and expel the air from the system.	
Connect the tubing to the Cannula.	
Understand the protocols for connecting to central lines/ portacaths etc	
Demonstrate use of the contrast warmer settings.	
Select an appropriate programme / protocol.	
Be able to adjust the programme settings as appropriate.	
Demonstrate the use of any delay settings as required.	
Know how to stop the injector in an emergency or contrast extravasation.	
Understand the implications of leaving air in the injector.	
In event of cardiac arrest, understand that the injector / connection tubing may be electrically unsafe during defibrillation.	
Demonstrate how to clean the injector.	
Disconnect the syringe and tubing after use with correct disposal techniques.	
Switch off the injector.	

I certify that the above person has been trained and is competent to use the equipment without supervision. I further certify that this list is not exhaustive and does not constitute comprehensive training.

Signed _____ Assessor _____ Date _____

I have received adequate training and I feel that I am competent to use this equipment without supervision.

Signed _____ Operator _____ Date _____